MY AIR. MY HEALTH.

MYALLERGY DETECTIVE™

ALLERGY AIR TESTING SYSTEM

TEST THE AIR IN YOUR
PROPERTY FOR THE TOP
ASTHMA AND ALLERGY
TRIGGERS INCLUDING
MOLD, POLLEN, ANIMAL
HAIR AND MORE.

INSTRUCTION GUIDE

UNIT FEATURES

Contact Us

customer@MyAllergyDetective.com | www.MyAllergyDetective.com

Thank you for purchasing MyAllergyDetective™. For best results, please read and follow instructions carefully. For the most up-to-date instructions, FAQs and videos, visit MyAllergyDetective.com

Testing Instructions Start Here: Customize your report

RECOMMENDED STEP (optional):

Take digital photos of your sampling locations in action. Upload your photos to your online report during the registration process. This will provide a more customized and documented report.

U.S. Patent | Made in China | Assembled in the U.S.A.

1-Year Manufacturer's Limited Warranty

STEP 1: TAKE AIR SAMPLES

a

Required Outdoor Air Sample:

- On one cassette, mark the box labeled "Outside Sample." (This will be your **REQUIRED** control sample.)
- Ensure you have a power outlet to plug in to. You may need an extension cord.
- Take sample when it's not raining or snowing, in an open area 10 ft. away from the structure and overhangs, 3 to 5 ft. off the ground. (i.e. Use a chair to set the pump on.)

b

- Remove both tape flaps from the top and bottom of the air cassette. (Save flaps for resealing cassettes.)
- Open the pump lid. (This will remain open during testing.)
- Insert cassette onto the air sample inlet tray and secure firmly in place.

- Push the button to start the sample, button will illuminate during testing.
- The pump will shut off automatically after the 5-minute sampling session is complete.


- Reseal top and bottom of air cassette with tape flaps after air sampling is complete.

Indoor Air Sample:

- Mark the remaining air cassette(s) with the room name. Samples should be taken in the middle of the room, 3 to 5 ft. off the ground. For additional tips on testing, visit MyAllergyDetective.com/FAQs.
- Repeat steps (b) through (d) outlined in Outdoor Air Sample section.
- Indoor sampling should be performed immediately after outdoor air sample.

Note: Do not reuse cassettes. Take indoor air samples within 15 minutes of the outdoor air sample. Outdoor sample must be repeated for all future testing sessions.

STEP 2: REGISTER ONLINE

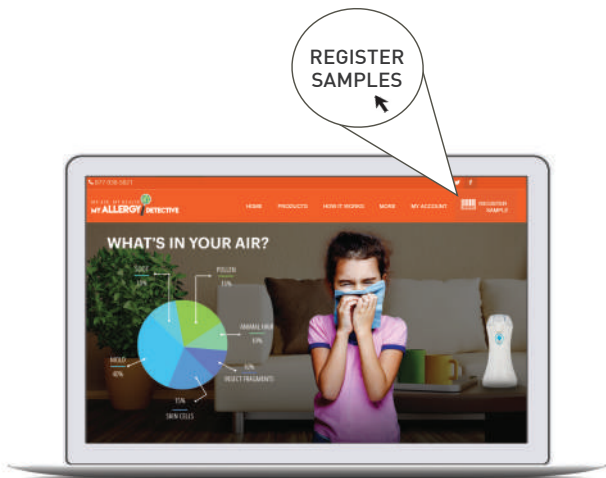- Visit [MyAllergyDetective.com](https://www.myallergydetective.com) to register your samples
- Click the “Register Samples” button at the top of the homepage and follow the instructions.
- Create a new account or log in.
- Complete payment for the lab analysis and note your confirmation number.

STEP 3: MAIL SAMPLES

a

- FedEx Express Shipping: 1-2 days to reach the laboratory

FedEx Express Shipping is trackable and convenient. For your convenience, call 1.800.GOFEDEx to schedule a free FedEx pick up from your location, Monday through Friday. You may also dropoff at any FedEx location.

b

- Insert air samples into the FedEx mailing envelope provided and mail to laboratory.

Note: Sending more than one envelope or package may cause samples to arrive to lab separately and cause possible delays with analysis. Please consolidate all samples into one envelope.

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CARACTERÍSTICAS DE UNIDADES

Contáctenos

customercare@myallergydetective.com | www.MyAllergyDetective.com

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